



# **EQUIPMENT INFORMATION SHEET FOR MS120105 – DISPOSABLE FACE SHIELD**

## **Safety Precautions**

Use the following safety guidelines to help protect your equipment from potential damage and to ensure your own personal safety.

- PPE Category I, manufactured in accordance with the essential health and safety requirements of the Directive, in accordance with BS EN 166:2002, CE Marked.
- Adjustable, elasticated headband. Always ensure that the headband has been adjusted correctly, using the holes provided, to maintain a secure fit during use. Please note the elastic may contain Latex.
- Foam forehead band for a comfortable fit.
- The face shield has a large viewing area suitable for use over glasses, prescription lenses and protective masks. It can be used for prolonged periods with no colour distortion.
- Regular cleaning using warm water and soap is required. The use of solvents or harsh abrasives is not recommended.
- The face shield should be inspected before each use to ensure that it is not damaged and can be secured correctly. Any defective face shield should be discarded immediately and replaced. It is not possible to repair a face shield.
- As the face shield may become contaminated during use, it cannot be recycled. Please dispose of appropriately.
- Individually packed in a sealed polythene bag. This provides protection against dust and contamination before use. Always store safely away from objects that damage the bag or face shield.
- Provides protection against sprays and splashes of non-corrosive fluids only.

## **Marking**

The face shield is marked as follows in accordance with EN 166:

A B C – DD E  
(ELMA 1 O – EN166 3)

Where:

A – Manufacturer Identification  
(Elma Electronic UK)  
B – Optical Class  
(Optical Class 1)  
C – Original Or Replacement Item  
(Original Item)

DD – Applicable Standard  
(BS EN 166:2002)  
E – Symbol For Field Use  
(Liquid Droplets Or Splashes)

## **Product Faults**

This product has been supplied free from defects, however should a fault arise please contact us for details of the replacement and warranty procedure.